








Operating Instructions

1. Insert a D-piece into the instrument and fit a new cardboard mouthpiece.
2. Press and release the button to turn on the device and to begin a breath test. As you do this, take a deep breath and hold it. The orange and red indicators will light up.
3. Continue to hold your breath until the indicators count down from 6 to 2 (this takes approximately 15 seconds).
4. When the green indicator flashes, start to blow slowly into the mouthpiece, aiming to empty the lungs completely.
5. The instrument will then display the level of CO by the relevant indicator light.
6. To turn the device off, press and hold the button for 3 seconds or wait for approx. 30 seconds after the end of a test when the unit will switch off automatically.
7. Switch the device off and back on again to start a new breath test.
8. To test the battery, press and hold the button for 3 seconds while the instrument is off. The status of the battery will be displayed by the indicators: **green** - good battery, **orange** - low battery, **red** - very low battery, replace device.

Results

		ppm	%FCOHb
		>51	5.66
6		35-50	
5		26-35	
4		21-25	
3		7-20	1.98-5.66
2		4-6	1.13-1.70
1		0-3	0-0.85

Bedfont Scientific Ltd,

105 Laker Road, Rochester Airport Industrial Estate,
Rochester, Kent, ME1 3QX, England

Tel: +44(0) 1634 673720 Fax: +44(0)1634 673721

E-mail: ask@bedfont.com www.bedfont.com

© Bedfont Scientific Limited 2010

Issue 1 - June 2010, Part No: LAB318_baby

Bedfont Scientific Limited reserve the right to change or update
this literature without prior notice.

Registered in: England and Wales. Registered No: 1289798

