



Bedsoft Update Tool

Quick Start Guide

AUTHOR
Adam James Lewis
Technical Administrator

SIGNED 
DATE 02/08/2010

BEDFONT SCIENTIFIC LTD
105 Laker Road,
Rochester Airport Industrial Estate,
Rochester,
Kent ME1 3QX England

Tel: +44 (0) 1634 673720
Fax: +44 (0) 1634 673721

Amendment Record

Date of Amendment	Document Issue	Pages Affected	Reason for Amendment
02/08/2010	1	All	<ul style="list-style-type: none">• First Issue••••••

Welcome to the Bedfont Scientific Update Tool quick start guide.
Here you will learn how to locate, install and operate the Unlock Tool.

First you will need to locate the software link 'Bedsoft V2.0. Update Tool (GastroChart: V2.xx.xx – COdata+: V2.xx.xx)' on the Tech Support page.

Click the link to begin the download.



Tech Support

Product & Technical Support




Interactive Downloads (the videos may take a few minutes to download)

- [Remote Helpdesk Support Tool](#)
- [piCO+ Training Video \(Windows\)](#)
- [piCO+ Training Video \(Flash\)](#)
- [NiCOdep® Test](#)
- [Fagerström Test](#)
- [Bedsoft V2.0 Update Tool \(GastroCHART:v2.00.44 - COdata+:v2.00.39\)](#)
- [Calibration Unlock Tool \(Micro+, PiCO+, Gastro+ & ToxCO\)](#)

Select 'Run' to install the program now or save to install at a later date.


(I have chosen 'Run').

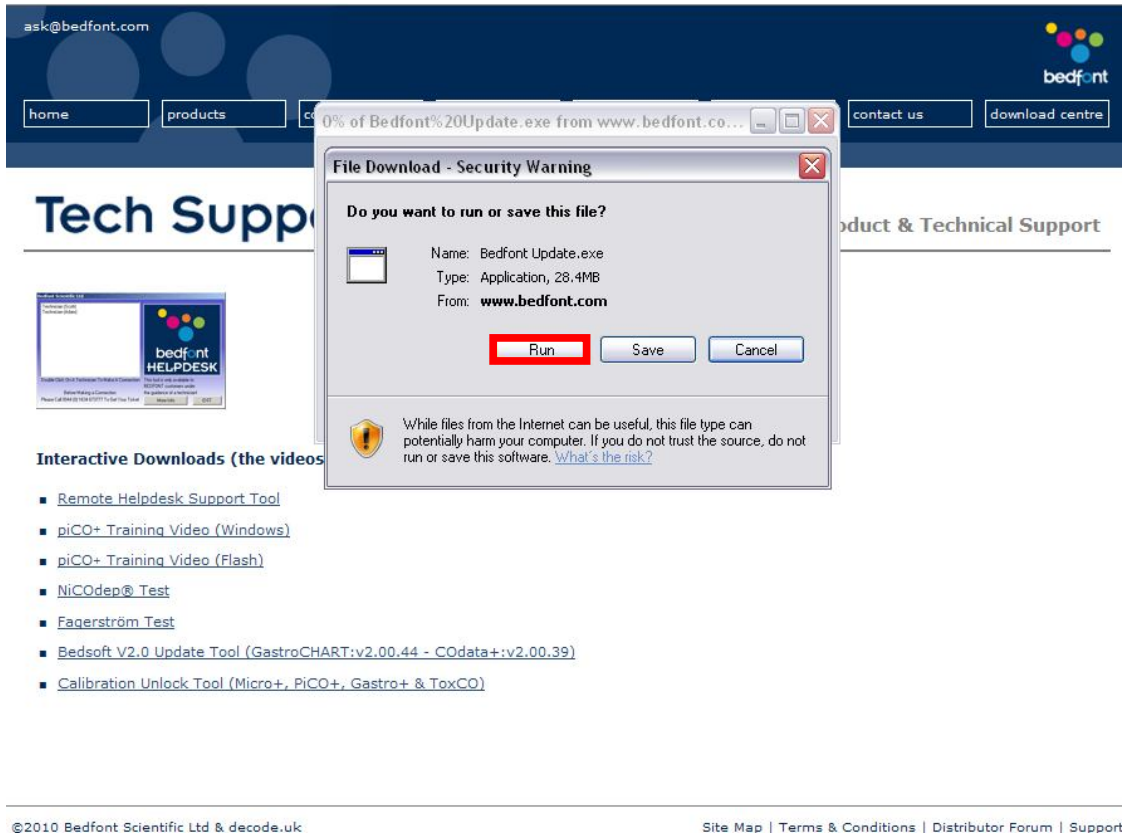


Then select 'Run' again to start the installation wizard.

The screenshot shows the Bedfont website's 'Tech Support' page. A security warning dialog box from Internet Explorer is overlaid on the page. The dialog box title is 'Internet Explorer - Security Warning'. The main text reads: 'The publisher could not be verified. Are you sure you want to run this software?'. Below this, it shows 'Name: Bedfont%20Update.exe' and 'Publisher: Unknown Publisher'. There are two buttons: 'Run' (highlighted with a red box) and 'Don't Run'. Below the buttons, a shield icon with a red 'X' indicates a warning: 'This file does not have a valid digital signature that verifies its publisher. You should only run software from publishers you trust. [How can I decide what software to run?](#)'. At the bottom of the dialog, a SmartScreen Filter icon with a green checkmark indicates: 'SmartScreen Filter checked this download and did not report any threats. [Report an unsafe download.](#)'. The background website shows a navigation menu with 'home', 'products', 'contact us', and 'download centre'. The main content area has a 'Tech Support' header and a list of 'Interactive Downloads (the view...)' including: Remote Helpdesk Support Tool, piCO+ Training Video (Windows), piCO+ Training Video (Flash), NiCOdep® Test, Fagerström Test, Bedsoft V2.0 Update Tool (GastroCHART:v2.00.44 - COdata+:v2.00.39), and Calibration Unlock Tool (Micro+, PiCO+, Gastro+ & ToxCO). The footer contains '©2010 Bedfont Scientific Ltd & decode.uk' and 'Site Map | Terms & Conditions | Distributor Forum | Support'.


Click 'Next' to continue with the installation.

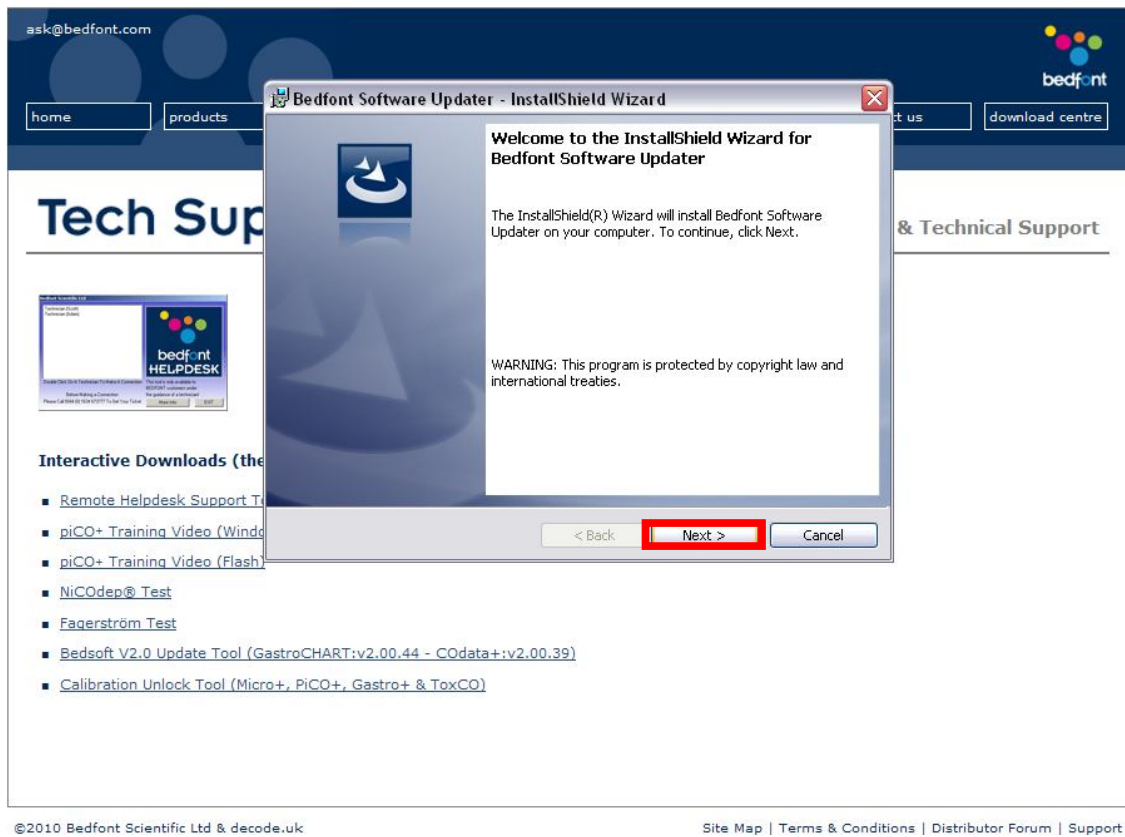


Type your user name and the name of your organization and select next.

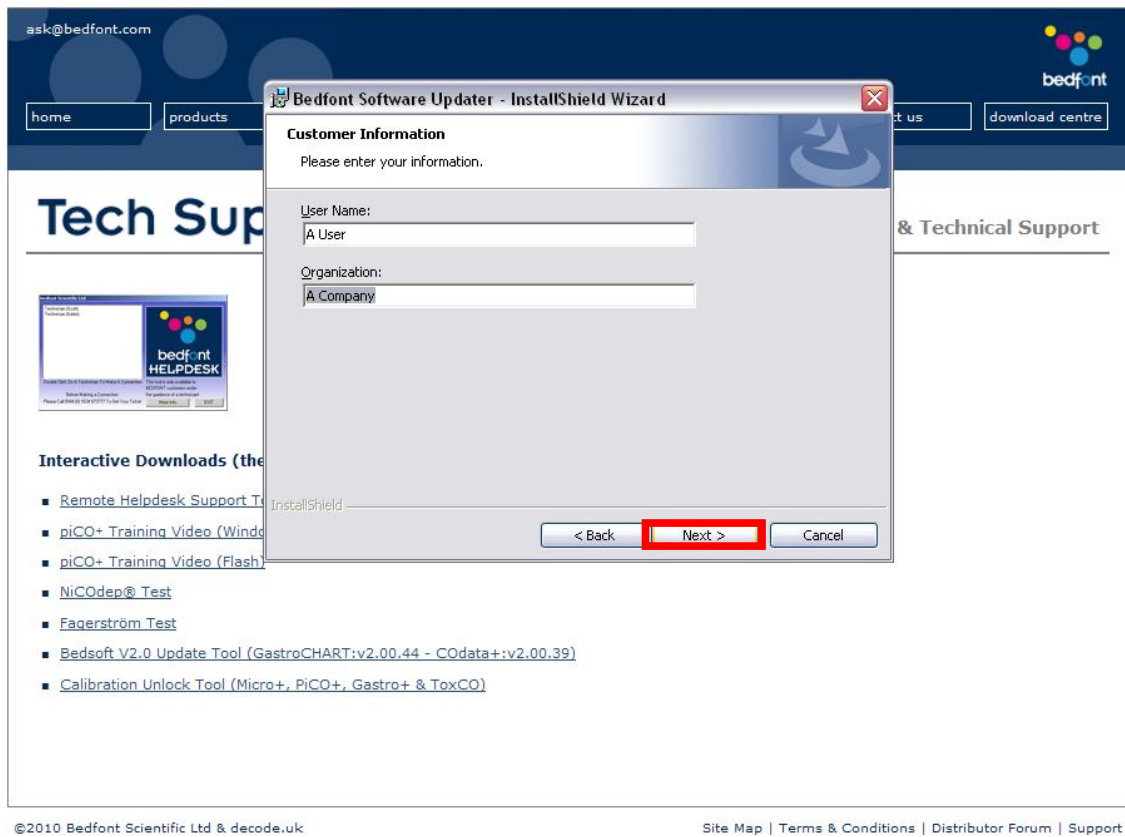

Select 'Install' to commence installation.

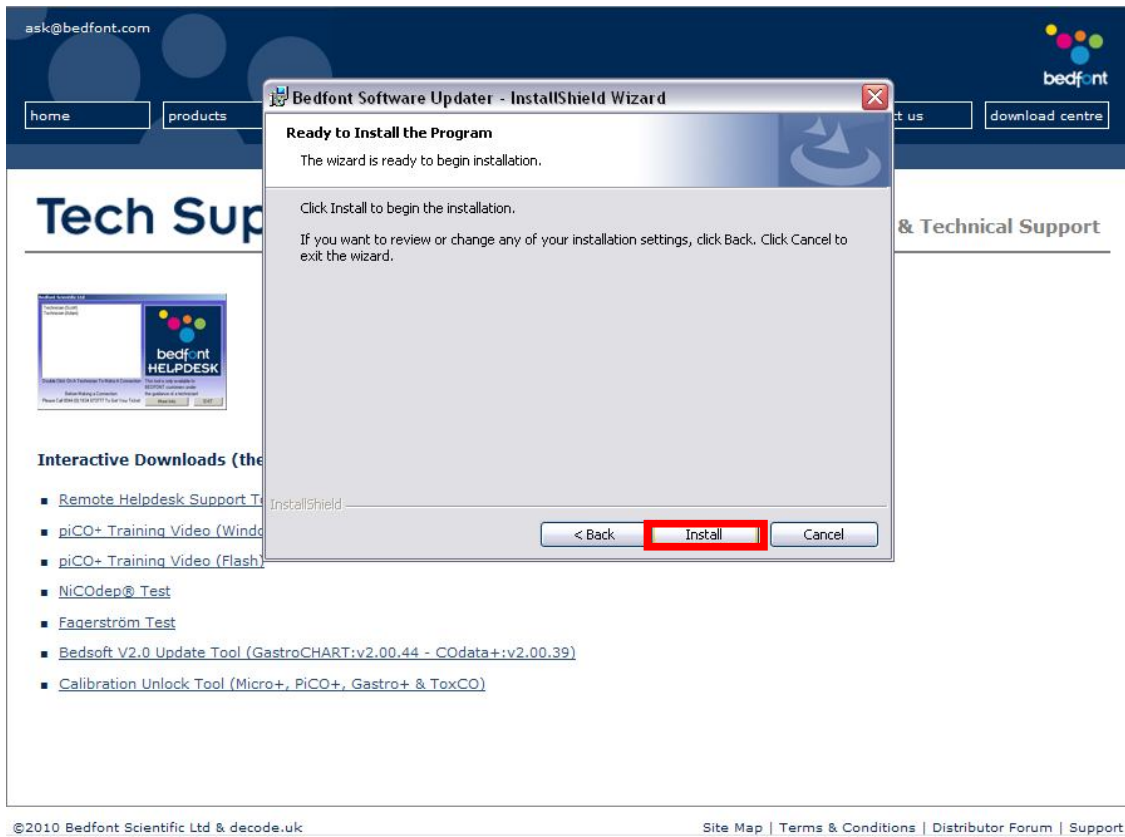


Once the installation is successful, select 'Finish' to complete the setup.

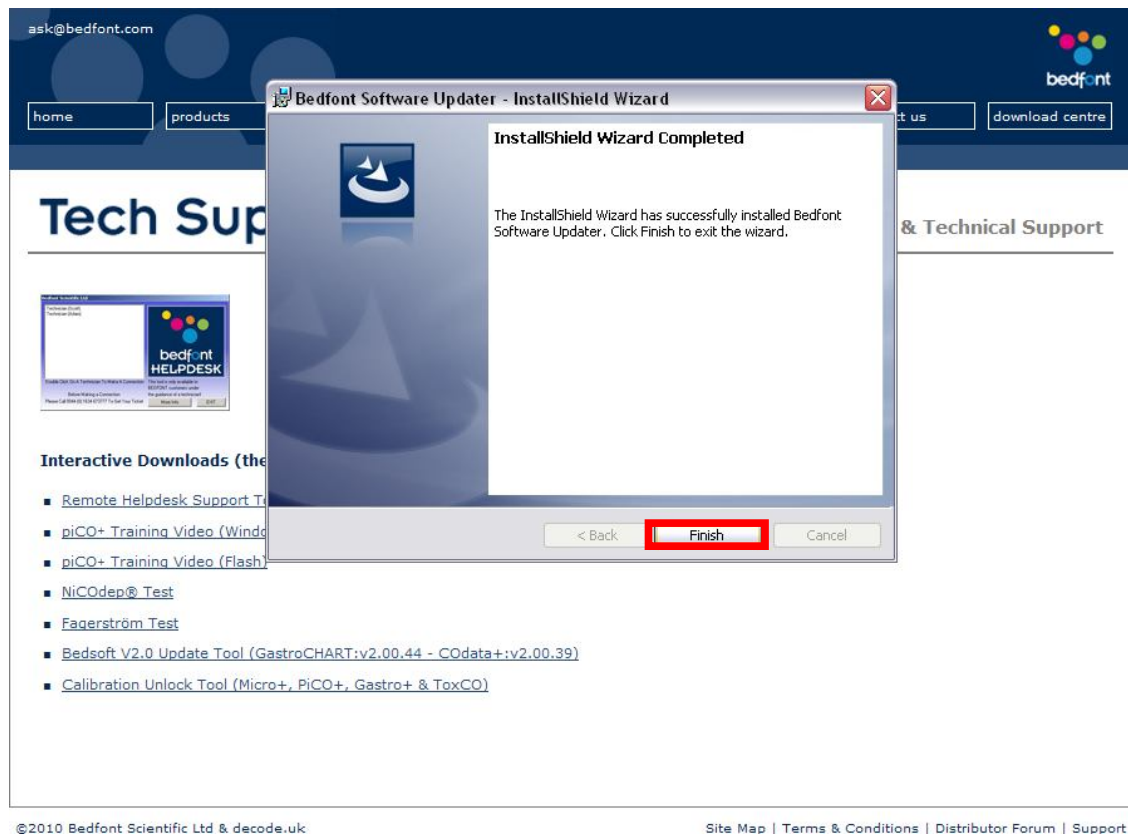


To run the software you will have an icon on your desktop which looks like the one below, double click it to run the software.




You will be presented with the window below, click the positive symbol '+' which relates to the software you wish to update (blue COData+, purple GastroCHART).

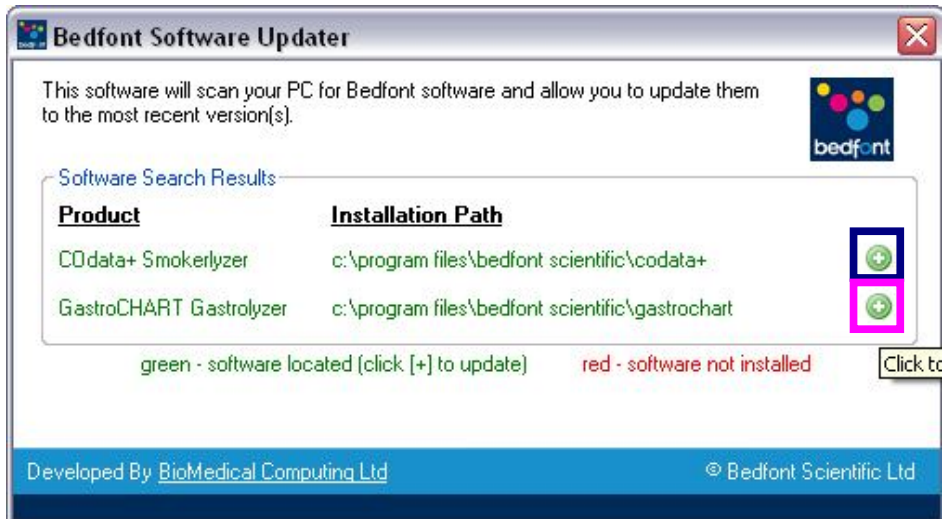

Please ensure COData+/GastroCHART are not currently active on your system and select 'OK' to continue with the update.

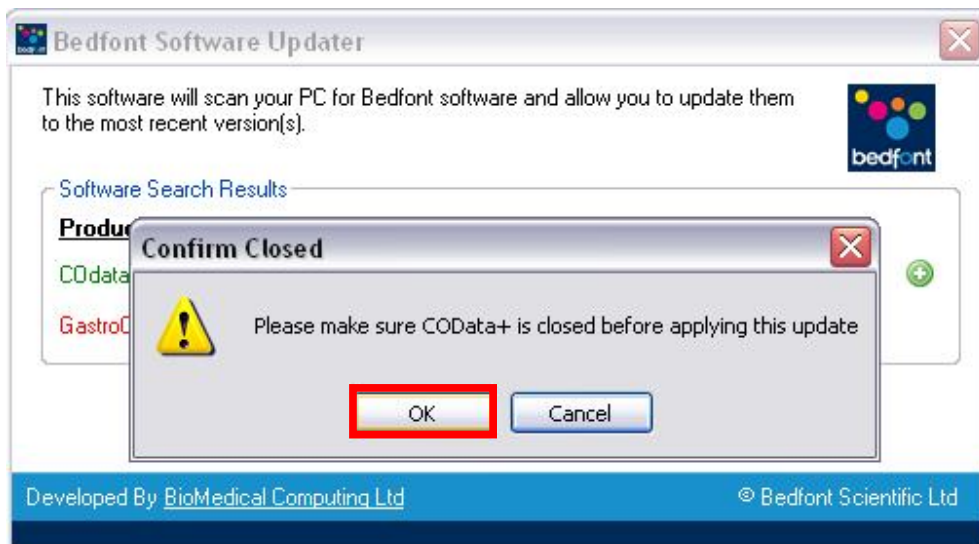

Once the update is complete you will be notified via the following window, click 'OK' to remove this message and close the update tool using the close button in the top right of the window.

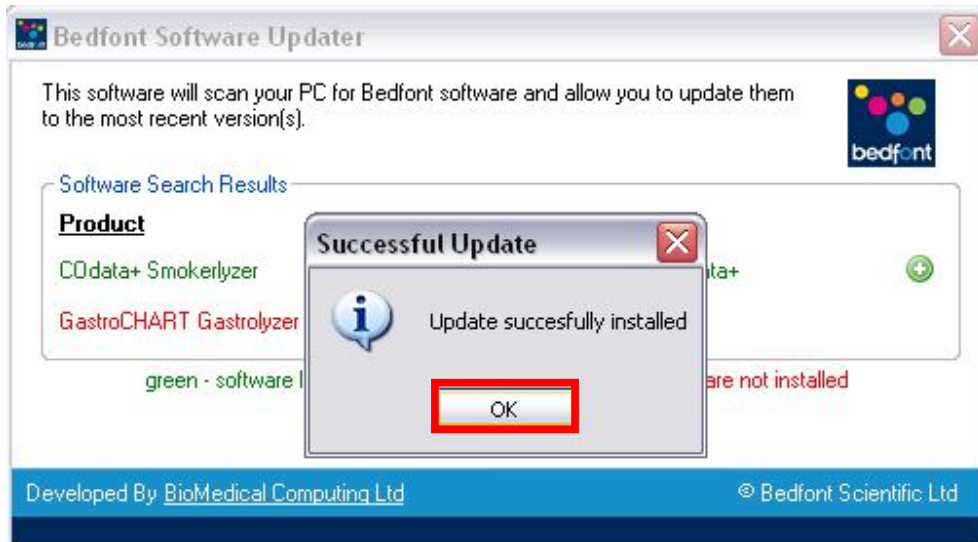

Your software is now up to date and you can continue to use it as before, thank you for using Bedfont's update service!